

# RP 360° Suture Passer




The jaw of the RP 360° suture passer may be rotated to four discrete positions (at 90° increments), relative to the handle, by grasping the red hub and twisting to the desired location. As the hub is rotated from one position to another, the user will feel the hub 'snap' into place, indicating it is in the desired position. The laser mark on the red hub indicates the orientation of the jaw opening. The jaw of the RP 360° may be rotated relative to the handle at any time during the use of the device, including after the suture passer is in the joint and suture is loaded.



#### Step 1:

Load the RP 360° flexible needle (3910-900-091) into the RP 360° suture passer reusable instrumentation handle (3910-900-090). Insert the needle into the cannulated shaft with the fin facing up. Push the needle forward until the base of the fin drops into the slot, and ensure that the needle knob is fully seated in the slot of the suture passer handle.

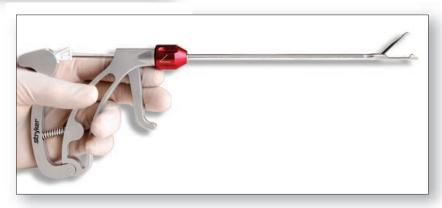






# Step 2:

Grasp the distal tip of the passing suture, and slide the suture into the V-shaped slot in the lower jaw, leaving the free end of the suture between the upper and lower jaws. When properly loaded, the suture will snap into place. Improved suture management may be achieved if the free end is 2cm to 3cm long.

## Step 3:

Close the jaws and introduce the RP 360° suture passer to the surgical site. If using arthroscopic surgical techniques, use at least a 6.5mm cannula. When ready to pass the suture, open and position the jaws of the suture passer at the desired location. Close the jaws by gently squeezing the small lever with the first two fingers.

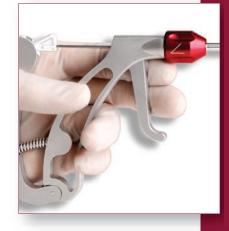



#### Step 4:

To deploy the needle and suture, squeeze the handle with the palm while maintaining the small jaw lever in the closed position. After passing the needle and suture, release the grip on the handle to allow the needle to fully retract, while keeping the jaw closed to maintain a good grip on the tissue.





## Step 5:

Grasp the loop of suture that was passed through the tissue with a hook or grasper from another portal. Close the jaws and remove the suture passer from the cannula. Alternatively, the loop of suture may be grasped using the jaws of the suture passer.



# Step 6:

After completion of the surgery immediately remove and discard the disposable needle.

Do not attempt to re-process the needle.

If you have further questions, please contact your local Stryker Orthopaedics sales representative.

# Ordering Information 3910-900-090 RP 360° Suture Passer Instrumentation (Reusable) 3910-900-091 RP 360° Suture Passer Flexible Needle (Disposable)\*

본 Stryker 카다로그는 전 세계 공용이므로, 본 카다로그에 등재된 제품들 모두가 국내 KFDA 품목허가/신고가 된 것은 아닐 수 있습니다. 이에 제품의 허가 등의 여부를 반드시 사전 확인하시기 바라며, 관련 문의는 한국스트라이커(주)(대표전화 02-565-7303)로 연락주시기 바랍니다.

The information presented in this brochure is intended to demonstrate a Stryker product. Always refer to the package insert, product label, and/or user instructions before using any Stryker product. Products may not be available in all markets. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.



Stryker Korea 서울시 강남구 삼성동 158-24 동성빌딩 11F

Tel: 02-565-7303 Fax: 02-565-3026 www.stryker.co.kr

<sup>\*</sup>Packaged in boxes of 5.