


ReelX STT™

Knotless Anchor System

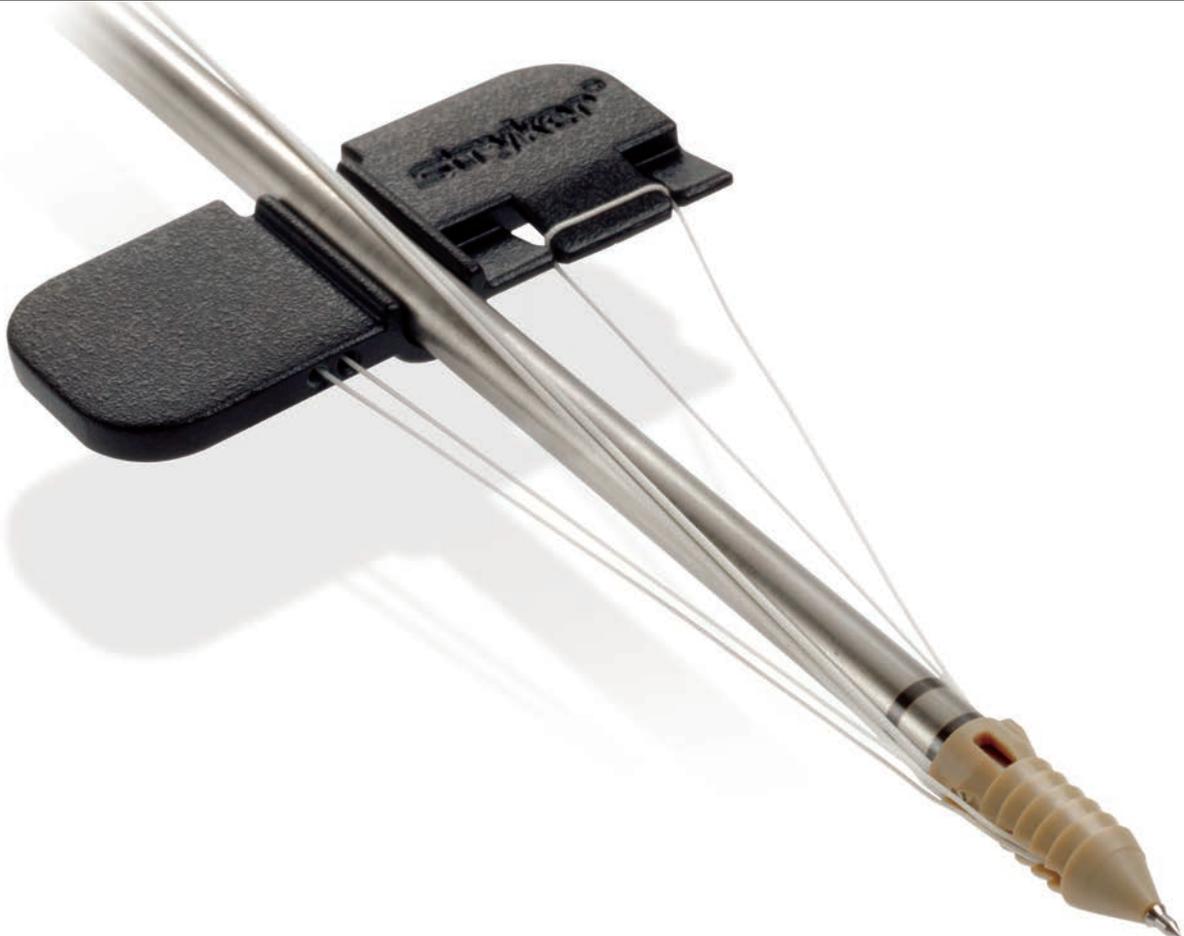

ReelX STT

Knotless Anchor System

The ReelX STT 5.5mm knotless anchor features an incremental tensioning mechanism, which enables patient Specific Tissue Tensioning during arthroscopic rotator cuff repair. The body of the ReelX STT has a diameter of 5.5mm, and with each incremental turn of the handle, expands up to 6.5mm. The expanding body of the ReelX STT is designed to provide enhanced fixation.

Simplicity was important when designing the ReelX STT. Working suture is loaded through the use of a pull tab that is securely attached to the inserter handle. The ReelX STT also features a sharp metal tip, making instrumentation to aid in insertion optional.



Quick Steps:

Load up to two #2 sutures through the anchor using the suture threader.



While maintaining slight tension on the suture, slide the anchor down on the bone to the desired insertion site. Remove any excess suture between the anchor and tissue by gently pulling on the ends of the suture. Mallet the anchor in until it is flush with the cortical surface of the bone.



ReelX STT

Knotless Anchor System



Features and Benefits

- Anchor Body Expands from 5.5mm up to 6.5mm
- Sharp Tip Makes Instrumentation Optional
- Incremental Tensioning Allows for More Control Over The Repair

Ordering Information


Part Number	Description
3910-600-060	ReelX STT Knotless Anchor*
3910-004-040	6.5mm Zip Drill (4.0mm x 13.5mm)
3910-003-065	6.5mm Tapered Awl

* Packaged in Boxes of 5

Release the tether suture from the inserter handle.



While holding the yellow end of the handle, rotate the black knob clockwise a minimum of one revolution to spool excess suture into the anchor. A maximum of three complete revolutions can be made. The implant has one locking point every 60° of revolution of the black knob, and advances approximately 1.5mm of suture for every 60° of rotation. Additional tension may be applied to the suture by re-engaging the inserter shaft.

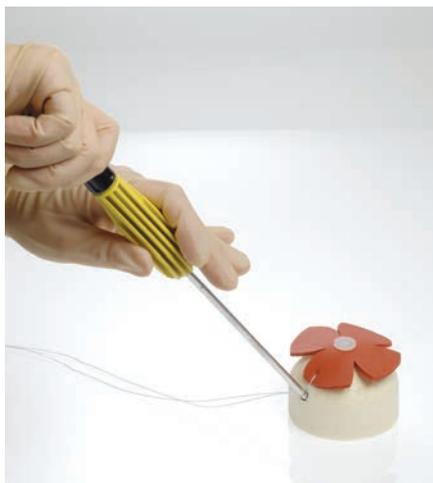

Joint Replacements

Trauma, Extremities & Deformities

Cranio-maxillofacial

Spine

Biologics

Surgical Products

Neuro & ENT

Interventional Spine

Navigation

Endoscopy

Communications

Imaging

Patient Care & Handling Equipment

EMS Equipment

본 Stryker® 카달로그는 전 세계 공용이므로, 본 카달로그에 등재된 제품들 모두가 국내 KFDA 품목허가/신고가 된 것은 아닐 수 있습니다. 이에 제품의 허가 등의 여부를 반드시 사전 확인하시기 바라며, 관련 문의는 한국스트라이커(주)(대표전화 02-565-7303)로 연락주시기 바랍니다.

The information presented in this brochure is intended to demonstrate a Stryker product. Always refer to the package insert, product label, and/or user instructions before using any Stryker product. Products may not be available in all markets. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.



Stryker Korea
서울시 강남구 삼성동 158-24
동성빌딩 11F
Tel : 02-565-7303
Fax : 02-565-3026
www.stryker.co.kr